

Instructions for Installing and Removing Scalpel Blades



1. To install the scalpel blade, align the blade with the scalpel handle as shown.

WARNING: Scalpel blades are extremely sharp. Always hold the back of the blade. Never force the blade. Use pliers to hold the blade securely if it becomes stuck on the handle.



2. Align the blade into the groove in the handle and gently slide it toward the scalpel handle.



3. Continue to slide the blade onto the scalpel handle until it locks in place.



4. To remove the blade, carefully lift the bottom of the blade until it unlocks from the scalpel handle as shown.



5. Slide the blade off the scalpel handle, moving it away from the handle as shown.